

EHVT-50 Software Update

Dear customers, we have released the new software version **V4.0.7** of EHBT-50, the main updated contents are as follows:

1. Testing:

1.1 Support for enhanced mode for feces and urine testing, with an enhanced mode indicator added to the report details.

1.2 Support for retesting, allowing users to flag missed parameters and send the data back to the IoT system.

1.3 Support for automatic recognition of 3-well and 2-well counting chambers, with pretreatment and testing performed according to the corresponding protocols (blood).

1.4 Optimization of feces and urine testing.

2. Report:

2.1 Optimized report layout added display of full names and adjusted corresponding export and print formats.

2.2 Added new blood parameters: agglutinated red blood cells, spiked red blood cells, NRBCs, and large and small lymphocytes.

2.3 Added support for AI interpretation in Spanish, Portuguese, Russian, and French.

2.4 For data transmitted from PCs, added display of modification notes, modifier, and time.

2.5 Changed the abbreviation for Bacillus in feces reports to: SFB.

2.6 The blue background in the RBC large image display will be grayed out.

2.7 Added language support for Romanian

3. Workbench:

3.1 Receives and displays data returned from Open Dx review.

3.2 Supports the deletion of orders and items within orders on the instrument side.

4. QC:

4.1 Support for uploading quality control data to the IoT and LIS.

4.2 Uploading individual quality control data records to the LIS.

4.3 Addition of the HCT parameter to quality control results.

This version was released on 2026/4/30. Please connect to the network and update the version ASAP. If the network at your site is poor, please contact us and we will send you the software package and guide you on the upgrade.