

EHVT-75

Software Update Notification Version 4.0.22

v4.0.22	2026-08-07	5 Updated	6
RELEASE	RELEASE DATE	MODULES	NEW FEATURES

Dear Customer,

To continuously improve our products and elevate your experience, we are pleased to release version **4.0.22** for the EHVT-75. The following are the main contents of this update.

Test

NEW CBC Test adds new parameters: AWBC, HEB#, HEB%, SLYM%, LLYM%

NEW FMA Test adds new parameters: GIAC#, GIAT#

NEW UMA & FMA support three-hole counting plate consumables

Report

NEW UMA&FMA cell diagrams have added images of suspected casts, etc.

OPTIMIZED Optimize AI Interpretation

QC

NEW Support EH Liquid QC

Setting

NEW LIS Upload

Upload the device serial number, consumable batch number, and operator information.

FIXED LIS Upload

Fix the issue where the LIS failed to upload the CBC histogram.

FIXED Unit

Correction of conversion coefficients for HGB, MCH, MCHC, HDW_SD in the Netherlands unit system.

Others

OPTIMIZED Optimize UI

This version has been officially released. You can connect your device to the network, wait for the software package to download, and after seeing the update prompt, proceed with the online upgrade.

If you are unable to upgrade online, you can download the software package from the following link:

https://drive.google.com/drive/folders/1xI7L894dakom2TeDgHuINQDf9Yo9e4kN?usp=drive_link

Pls contact the after-sales team to guide you to perform the upgrade via a USB drive.

Thank you for choosing Ozelle.

For any questions about this release, please contact the after-sale service team for support.