

EHBT-25

Software Update Notification Version 4.0.19

V 4.0.19	2026-05-26	1 Updated	5
RELEASE	RELEASE DATE	MODULES	NEW FEATURES

Dear Customer,

To continuously improve our products and elevate your experience, we are pleased to release version **4.0.19** for the EHBT-25. The following are the main contents of this update.

Settings

NEW Language

System language added - French

NEW Printer Connection


Supports Wi-Fi printers (compatible with IPP protocol), and adds the function of clearing cache.

NEW Printing setting

Thermal printing reports support horizontal and vertical orientations.

A4 printing reports support standard mode and minimalist mode.

Page layout options: Header + Footer, Header

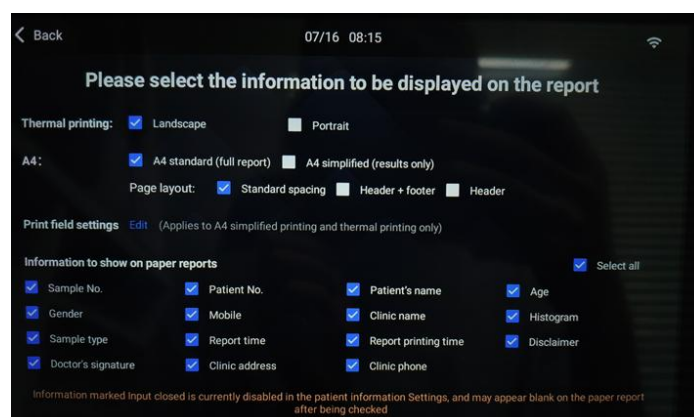

NEW Reference Range

Supports customer-defined grouping

Reference group	Gender	Lower age limit (>)	Upper age limit (<)
Default	Gender-neu...	16 Years	999 Years
Male	Male	16 Years	999 Years
Female	Female	16 Years	999 Years
Children	Gender-neu...	0 Years	16 Years

Note:
 * Preset groups cannot be deleted, sex cannot be changed, you can edit age range, group name, and reference values;
 * Custom groups: you can edit group information and reference values, and delete the group (its reference ranges are deleted as well);
 * For groups of the same sex, age ranges must fully cover 0-999 years with no gaps or overlaps;
 * If sex is not entered for a test, the "Not sex specific" group is matched by default, matching priority: same sex > not sex specific.

Buttons: Cancel, Add reference group, Save

NEW Report Viewing Address

Report Browsing Address

Please enter the following address in the PC browser to view the report:

<http://192.168.111.17:8000>

Note: The PC and the instrument must be connected to the same local area network for the report to be viewed normally.

Close

ADJUSTED Clinic Info

The clinic name, phone number and address can be added. The information will be displayed on the printed report.

Clinic information

Clinic name
Please enter clinic name.

Clinic phone
Please enter contact phone number

Clinic address
Please enter clinic address

Buttons: Cancel, Save

This version has been officially released. You can connect your device to the network, wait for the software package to download, and after seeing the update prompt, proceed with the online upgrade.

If you are unable to upgrade online, you can download the software package from the following link:

https://drive.google.com/drive/folders/1zj4fo-cllOyfKJZQELflnbvwwJZ4Nfsd?usp=drive_link

Pls contact the after-sales team to guide you to perform the upgrade via a USB drive.

Thank you for choosing Ozelle.

For any questions about this release, please contact the after-sale service team for support.