

EHBT-50 Minilab

Software Update Notification Version 4.0.9


V 4.0.9	2026-06-09	3 Updated	6
RELEASE	RELEASE DATE	MODULES	NEW FEATURES

Dear Customer,

To continuously improve our products and elevate your experience, we are pleased to release version **4.0.9** for the EHBT-50 Minilab. The following are the main contents of this update.

Testing

NEW CBC BC Method supports capillary (The sample volume only requires 30 microliters)

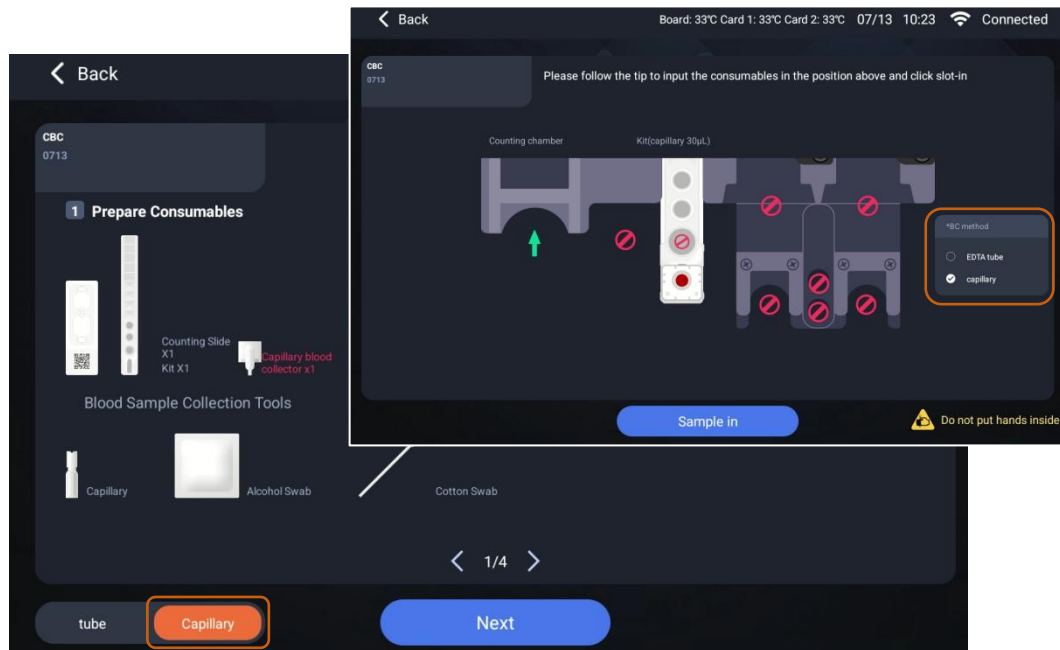

NEW CBC supports three-hole counting plate

The device will recognize the QR code on the counting plate, confirm the type of consumables to be used, and execute the corresponding process.

Quality Control (QC)

NEW Ozelle's self-developed liquid QC Material (EH) Support

< Back

Board: 35°C Card 1: 35°C Card 2: 35°C 07/13 10:26 Connected

* QC Material: ☐ SMX ☒ EH ☐ BIORAD

* Type ☒ CBC

* Batch: Click Input

ID: Click Input

* Level: ☐ L ☐ M ☐ H

* Expiry Date: YYYY/MM/DD

Item	Unit	*Target Value	Deviation Limit (±)
WBC	10 ⁹ /L	Enter Target	-
RBC	10 ¹² /L	Enter Target	-
HGB	g/L	Enter Target	-
MCV	fL	Enter Target	-
PLT	10 ⁹ /L	Enter Target	-

Dry quality control items do not require manual entry

Save

Settings


NEW Language

System language added - Spanish, French, Russian

NEW CBC Detection Enhanced Mode


If the “RBC Enhanced Detection Mode” is enabled: the detection of AWBC and SRBC will be increased. Positive samples will display the results of these parameters and the cell images. At the same time, the testing time will be extended.

If the “Malaria Enhanced Detection Mode” is enabled: Mode 1 indicates high sensitivity, while Mode 2 indicates high specificity. Customers can choose the corresponding mode according to their own needs.

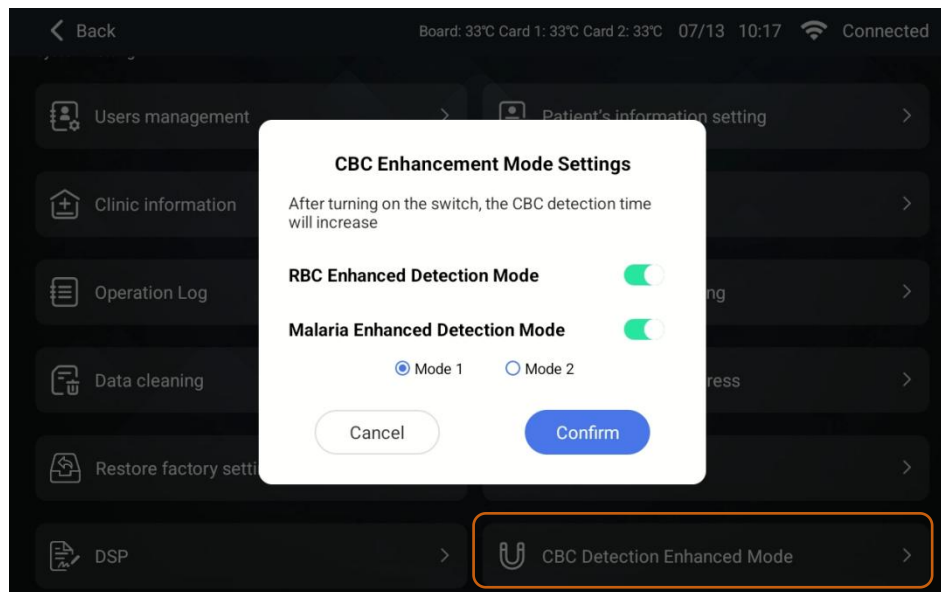

NEW PC Connection

After enabling this setting, it is possible to connect with the PC software OpenDX

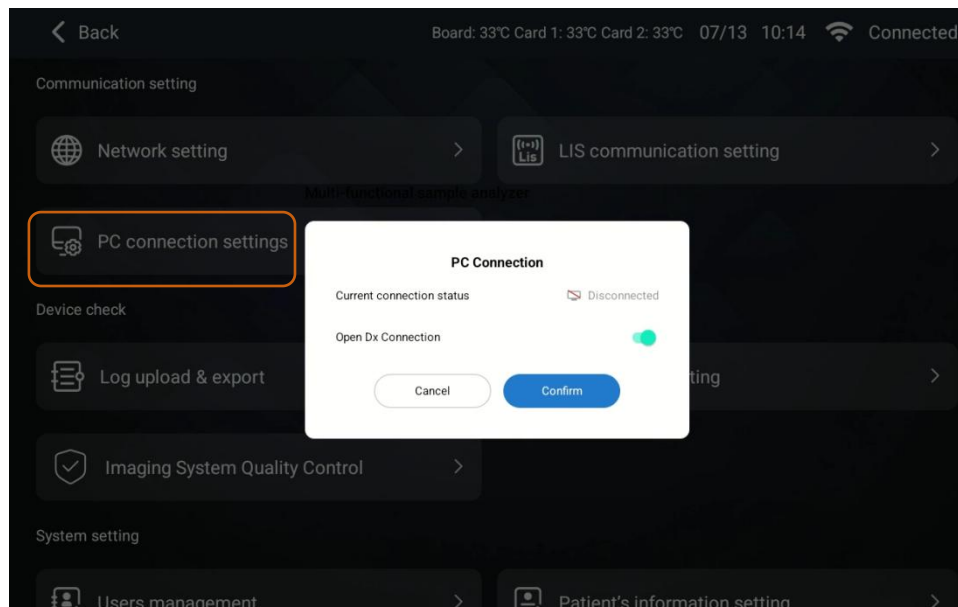

This version has been officially released. You can connect your device to the network, wait for the software package to download, and after seeing the update prompt, proceed with the online upgrade.

If you are unable to upgrade online, you can download the software package from the following link:

https://drive.google.com/drive/folders/1wAJSh0j4O2FxmET7fA-eqGG_IMpsHIHT?usp=drive_link

Pls contact the after-sales team to guide you to perform the upgrade via a USB drive.

Thank you for choosing Ozelle.

For any questions about this release, please contact the after-sale service team for support.