

EHBT-75

Software Update Notification

Version 4.0.21

V 4.0.21	2026-07-02	5 Updated	5
RELEASE	RELEASE DATE	MODULES	NEW FEATURES

Dear Customer,

To continuously improve our products and elevate your experience, we are pleased to release version **4.0.21** for the EHBT-75. The following are the main contents of this update.

Testing

OPTIMIZED Optimize RET results

ADJUSTED CBC test EP tube sample volume changed to 70uL

Report

NEW French, Russian, Portuguese, Italian AI interpretation

OPTIMIZED Optimization of RBC histogram

Quality Control (QC)

NEW Add units to the dry/wet quality control LJ graphs


Settings

NEW Language

System language added - Indonesian


NEW PC connection

After enabling this setting, it is possible to connect with the PC software OpenDX.

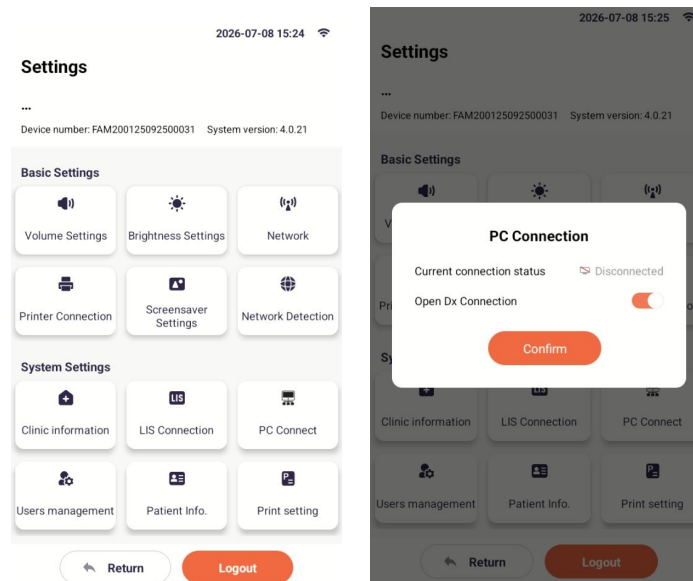

ADJUSTED LIS Connection

LIS connection supports both server and client connection modes

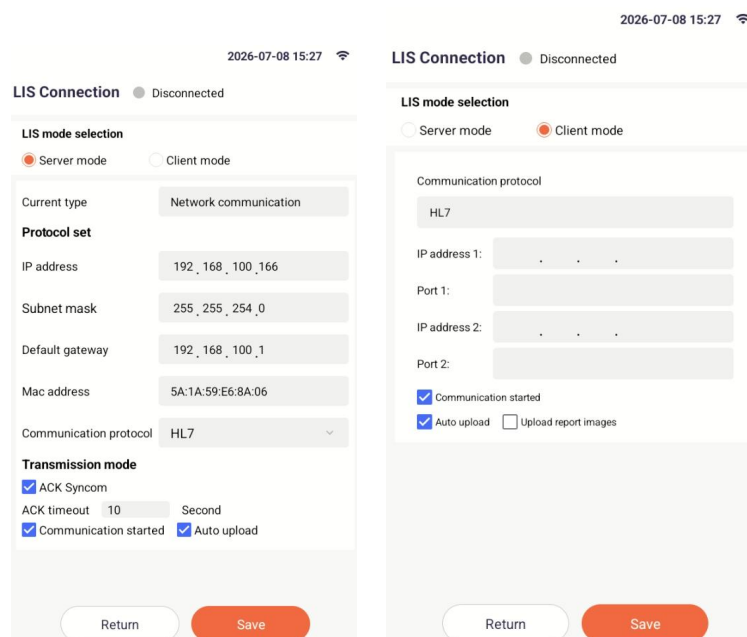

OpenDX

NEW Automatic Upload

When the analyzer reconnects to the OpenDX, it automatically uploads the last 50 reports within the past 7 days.

ADJUSTED Manual Upload

Manual upload of reports to openDX can upload a maximum of 10 at a time.

This version has been officially released. You can connect your device to the network, wait for the software package to download, and after seeing the update prompt, proceed with the online upgrade.

If you are unable to upgrade online, you can download the software package from the following link:

https://drive.google.com/drive/folders/1YQf6-a_88mhBrepUx-QR-qjzSTOCEFPk?usp=drive_link

Pls contact the after-sales team to guide you to perform the upgrade via a USB drive.

Thank you for choosing Ozelle.

For any questions about this release, please contact the after-sale service team for support.