

# EHVT-75

## Software Update Notification Version 4.0.14

|                 |                   |                  |              |
|-----------------|-------------------|------------------|--------------|
| <b>V 4.0.14</b> | <b>2026-06-15</b> | <b>5 Updated</b> | <b>10</b>    |
| RELEASE         | RELEASE DATE      | MODULES          | NEW FEATURES |

Dear Customer,

To continuously improve our products and elevate your experience, we are pleased to release version **4.0.14** for the EHVT-75. The following are the main contents of this update.

### Testing

#### **NEW** CBC new parameters added

AGG#、ACA#、NRBC#、NRBC/WBC%、 SLYM#、LLYM#


#### **NEW** CBC supports three-hole counting plate

The device will recognize the QR code on the counting plate, confirm the type of consumables to be used, and execute the corresponding process.

#### **NEW** UMA&FMA retesting function

#### **NEW** UMA&FMA Enhanced mode

Select the enhanced mode, and the device will capture more images. The testing time will be extended to 20-28 minutes.

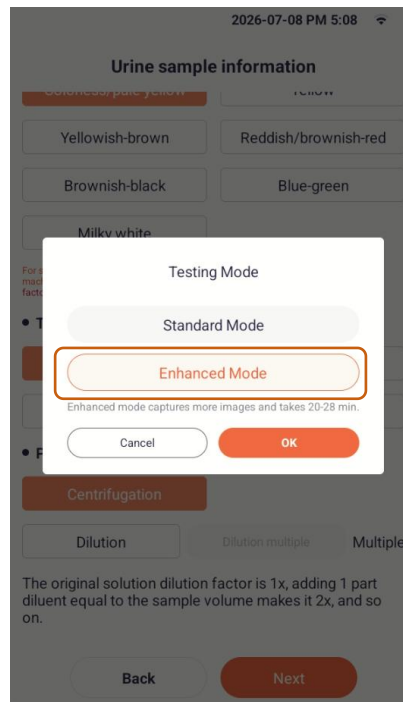

#### **OPTIMIZED** UMA&FMA optimization

Algorithm model optimization: Improve misidentification between major categories, increase dataset)

## Report

#### **ADJUSTED** Feline RET reference range

RET # 0 -  $30 \times 10^9/L$ ; RET% 0 - 0.34%

## Quality Control (QC)

#### **NEW** Automatic upload

QC results will be automatically uploaded to the BI system and LIS system.

**NEW** Manual upload to LIS system

2026-07-08 PM 5:11

No.:8 20251212-0046 High

Form **Dry** Parameters **CBC** Valid Until **2027.12.11**

| Param. | Upper Lim.<br>Target<br>Lower Lim. | Export<br>Upload LIS<br>Delete | SD<br>Mean<br>CV(%)   |
|--------|------------------------------------|--------------------------------|-----------------------|
| WBC    | 16.96<br>15.42<br>13.88            | 2026.05.26<br>16:21:50         | 0.01<br>15.63<br>0.09 |
| NST#   | 0.89<br>0.77<br>0.65               |                                | 0<br>0.76<br>0        |
| NSG#   | 10.28<br>8.94<br>7.6               |                                | 0.02<br>9.06<br>0.23  |
| NSH#   | 0.27<br>0.23<br>0.2                |                                | 0<br>0.24<br>0        |
| LYM#   | 4.79<br>4.16<br>3.53               |                                | 0.01<br>4.26<br>0.17  |
| MON#   | 0.89<br>0.77<br>0.65               |                                | 0<br>0.78<br>0        |

Current 1-2 of 2 records

Previous Next

Back

**NEW** New parameter HCT

**OPTIMIZED** Dry QC results optimization

## Settings

**NEW** Language

Italian, French, German, Traditional Chinese, Romanian

**NEW** Local factory account

factory/factory1234

## OpenDX

### NEW Manual Review

Supports users to manual review and mark undetected parameters.



This version has been officially released. You can connect your device to the network, wait for the software package to download, and after seeing the update prompt, proceed with the online upgrade.

If you are unable to upgrade online, you can download the software package from the following link:

[https://drive.google.com/drive/folders/1unXa7FSr78i7Qyfgnz6oOcGtH03RoZ9Z?usp=drive\\_link](https://drive.google.com/drive/folders/1unXa7FSr78i7Qyfgnz6oOcGtH03RoZ9Z?usp=drive_link)

Pls contact the after-sales team to guide you to perform the upgrade via a USB drive.

**Thank you for choosing Ozelle.**

For any questions about this release, please contact the after-sale service team for support.