

## EHVT-50 CBC Test Procedure


1. Wear gloves before testing.
2. Hold NFC card to the NFC card reading area for 3 seconds to register consumable data.
3. Click "Consumables" to check corresponding consumable data.
4. Prepare all the consumables.

Please prepare the test sample, pipette, tips, EP tube, and other consumables.




5. Input the patient information and choose the CBC test mode. And test operation instruction can be viewed.




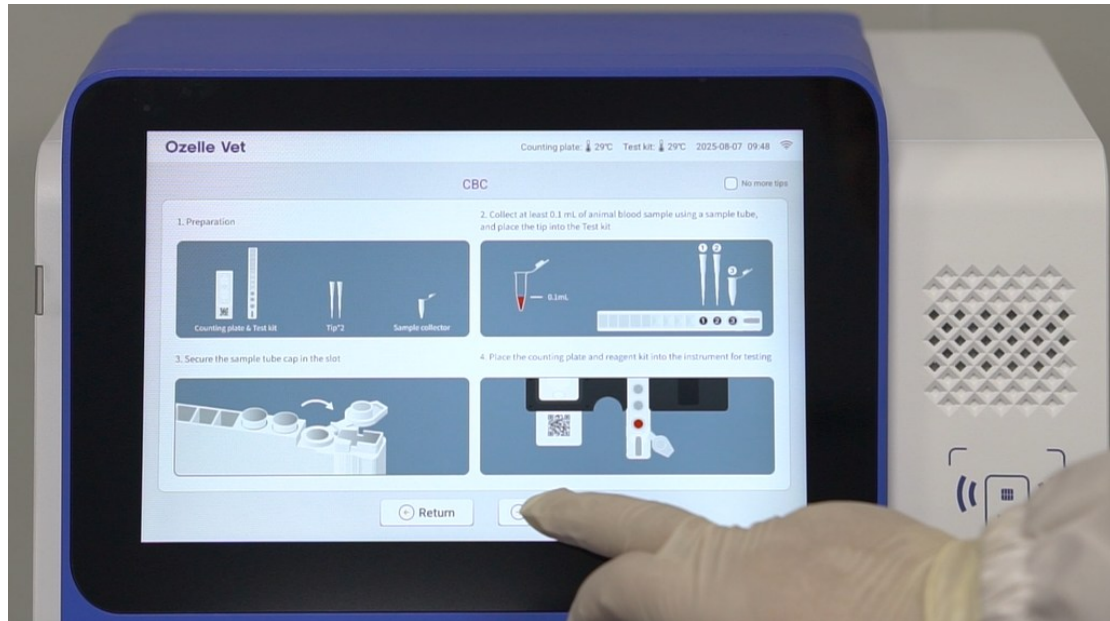


6. Click “Next” to the consumable inserting page, insert the counting chamber and test kit with two tips and tube with sample. And if the consumables are in the right position, the green icon will be on.

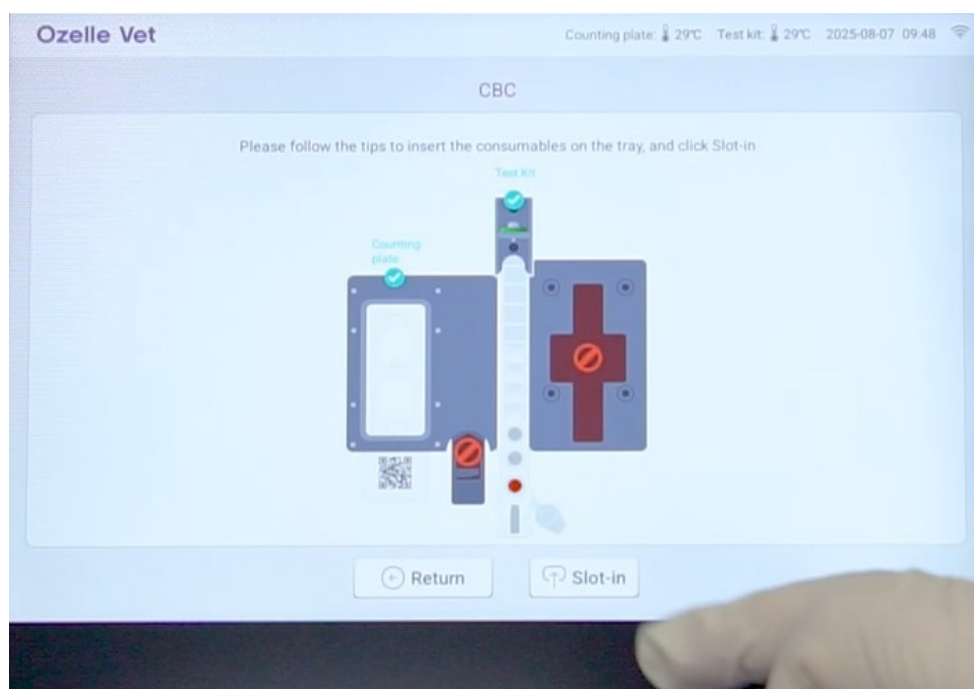

7. Click “Slot-in” to start test. During the process of testing, click the right bottom “Edit” to edit patient’s information.

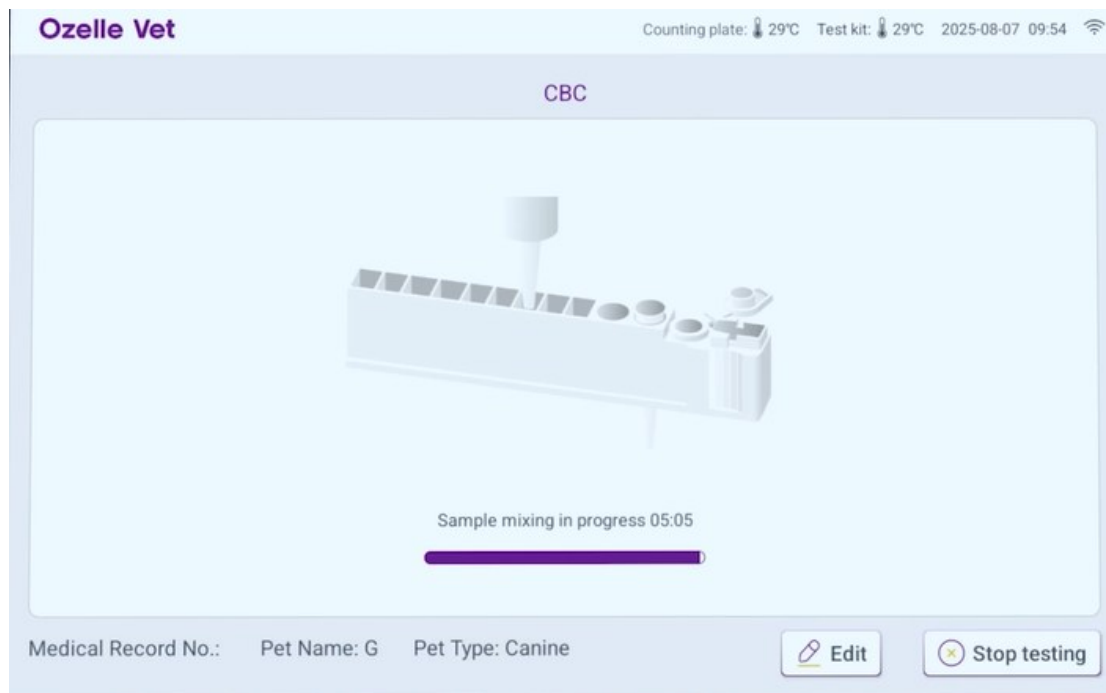

8. After testing, report could be viewed. After checking the report, remove sample or print report immediately.

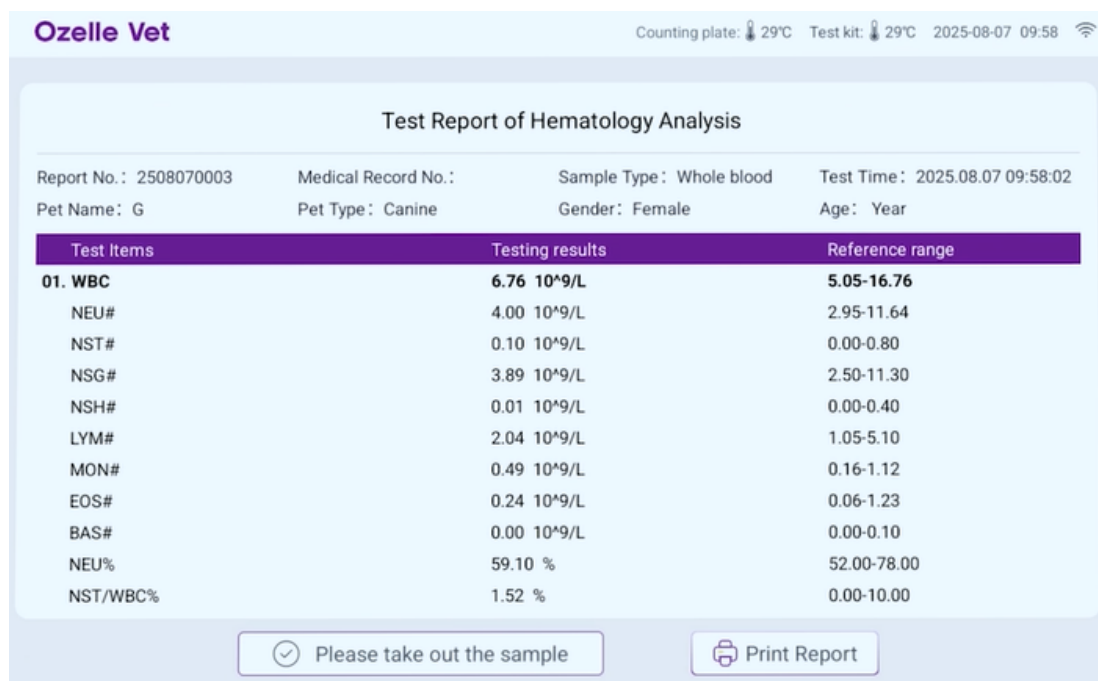

9. After removing the consumables, click “Removed” to finish the test.