

Update Points For EHVT-50

Dear customers, recently we have released the new version of EHVT-50, the version No is V4.0.19; the update points are shown below:

1. Quality Control (QC) Center optimization, including the addition of QC result export functionality. (Explanation: Can export the QC records)
2. Added spherocyte and ghost cell related information to blood reports, exported image packages, and report prints. (Explanation: We add spherocyte and ghost cell at the report)
3. Fecal test optimization and urine test optimization; And reduce Fecal test time to approximately 15 minutes.
4. Open the Factory mode to customer side, in case after we replaced the parts, need do factory QC. (account: factory password: factory1234)
5. LIS function modify: 1. Added content related to abnormal red blood cells to CBC reports; 2. Changed to upload using English units under all languages; 3. Added the configured hospital name to the LIS.

Please connect network then update the version ASAP. If the network at your point is poor, please contact us, we will send you software package, then guide you to update by USB flash.