

Update Point For EHVT 75

Overview

Dear customers, recently we have released the new version of EHVT 75, the version Num is V4.0.8; the update points are show like below:

1. AI report interpretation: Replace the interpretation content of blood red blood cells and white blood cells.
2. Compensate liquid aspiration volume according to the current temperature.
3. Fecal report: Revise the abbreviation of Giardia lamblia to GIA and the abbreviation of Ascaris eggs to TXE.
4. In LIS settings: Modify the option Power-on Auto-Connect to LIS to Auto-Connect to LIS and set it as checked by default.
5. Implement auto-reconnection for LIS after unexpected disconnection.

Please connect network then update the version ASAP. If the network at your point is poor, please contact us, we will send you software package, then guide you to update by USB flash.